

2

Collection Instructions **URINE**

CONTENTS

- Urine collection cup
- Disposable dropper
- Test tube
- Biohazard bag
- Absorbent pad



DO: Wash your hands before starting.



DO NOT pour liquid out of the tubes.



WAIT
Do not urinate for at least one hour prior to sample collection.

WARNING: Handle the tube carefully. Collection tube liquid can cause irritation if contact occurs with skin or other body parts. If contact occurs, immediately wash the affected area with plenty of soap and water for several minutes. If tube liquid gets in your eyes, immediately flush eyes with water for several minutes. If irritation occurs, seek medical attention.



If you've completed your **URINE** sample, you are ready to start the next step. See **BLOOD**.


INSTRUCTIONS




1
Open the cup package. To expand the cup, hold the bottom of the cup and pull.




3
Urinate into the cup an amount that's between the 10mL–50mL line marks on the cup. Place the cup on a hard, flat surface.

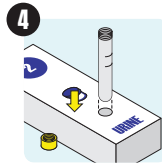


4
Open the **URINE** package. Remove the tube cap and place the tube in the hole on the top of the **URINE** box so it stands upright.

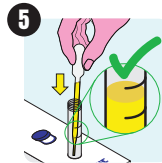

5
Using the pipette, transfer the urine from the cup into the urine tube until the fluid level is between the two black lines.

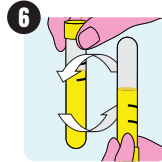

6
Remove the tube carefully from the box. Tightly recap the tube to prevent leaks. Invert the tube 5 times to mix.

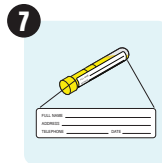

7
Write your personal information, date and time of collection on the **URINE** label, and place the label over the blue tube label.



8
Place the tube inside the biohazard bag with the absorbent pad.